AT04608

**INSTRUKCJA UŻYTKOWNIA, PAS BRZUSZNY
USER MANUAL, ABDOMINAL BELT
BEDIENUNGSANLEITUNG, BAUCHGURT
NÁVOD K POUŽITÍ, BŘIŠNÍ PÁSEK
NÁVOD NA POUŽITIE, BRUŠNÝ PÁS
MANUEL D'UTILISATION, CEINTURE ABDOMINALE
GEBRUIKERSHANDLEIDING, BUIKBAND
MANUAL DE USO, CINTURÓN ABDOMINAL
ISTRUZIONI D'USO, CINTURA ADDOMINALE
ANVÄNDARHANDLEDNING, BUKBÄLTE**

MD

CE

PL

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EN

Thank you for purchasing our product. Please read the user manual carefully. Do not use the product before familiarizing yourself with and understanding this manual. If you do not understand the warnings, notes, and recommendations, please contact a healthcare specialist or the seller to avoid product damage or personal injury.

WARNING: Check all parts for damage caused during transportation. If any damage is detected, DO NOT use the product. For more information, contact the seller.

TARGET PATIENT GROUP

Individuals dealing with diseases, dysfunctions, or injuries, for whose treatment/rehabilitation or compensation this product is intended (see the section regarding the purpose of the product in this user manual). The product can be purchased by the user independently or based on recommendations from a doctor, therapist, or another specialist. Whether the product is purchased independently or recommended by a doctor/therapist/other specialist, it is important to consider available sizes/necessary features and variants of the product, indications and contraindications for use, as well as the information provided by the manufacturer.

USE/INDICATIONS

The abdominal belt is an orthopedic product that provides effective support for the abdominal area, back, and lumbar muscles. It can be used prophylactically to support proper posture and protect the spine from excessive strain. The Velcro fastening allows for adjustment. It is most commonly recommended for abdominal hernias, weakened abdominal muscles, post-childbirth, post-abdominal surgeries, and after surgical procedures.

The abdominal belt strengthens the abdominal muscles and prevents displacement of internal organs due to abdominal hernias. The Velcro fastening allows for wide adjustment, ensuring a perfect fit to the body. The belt is recommended for abdominal hernias, weakened abdominal muscles, post-childbirth, post-abdominal surgeries, and after surgical procedures. Thanks to its sturdy design and easy adjustment, the belt supports proper posture and protects the spine from excessive strain.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe use of the product.

Skin diseases: Any inflammations, ulcers, wounds, or skin irritations in the area where the belt is to be worn may prevent its use.

Circulatory system diseases: In some circulatory disorders, such as circulatory insufficiency, prolonged pressure may be contraindicated.

Breathing problems: Individuals with respiratory issues, such as asthma, may experience discomfort due to the pressure of the belt on the chest and abdomen.

Gastrointestinal disorders: Some individuals with gastrointestinal diseases, such as acid reflux or irritable bowel syndrome, may experience discomfort or worsening of symptoms.

Intolerance to pressure: Individuals who feel discomfort or pain related to pressure may have difficulty tolerating the belt.

Some post-surgical conditions: In some cases, a doctor may decide that the belt is not recommended post-surgery due to the risk of complications.

LIST OF COMPONENTS

Abdominal belt, user manual

FEATURES AND DIMENSIONS

The belts come in two heights: 24 cm and 32 cm, in 7 sizes: XS, S, M, L, XL, XXL, XXXL

Size (waist circumference):

XS - <80 cm

S - 80-90 cm

M - 91-100 cm

L - 101-110 cm

XL - 111-120 cm

XXL - 121-140 cm

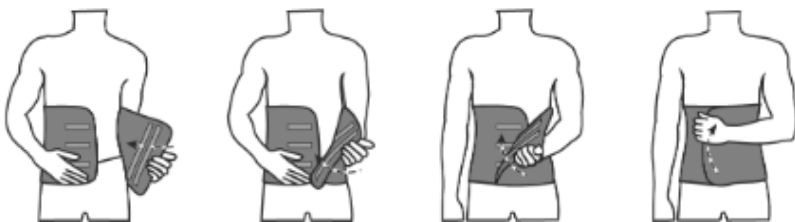
XXXL - >140 cm

Material: Polyamide (Pa) 15%, Polyester (Pes) 75%, Elastane (EA) 10%

USER INSTRUCTIONS

Detach the Velcro. The belt is placed on the lumbar-sacral region of the spine, and then fastened with Velcro at the front.

The product has a wide range of adjustability. The product can be quickly and easily adjusted to fit the patient's body shape. The Velcro fastening ensures effective immobilization.

NOTES AND RECOMMENDATIONS

Please read this manual carefully before using the product.

Before each use, ensure that the product has no fraying at the seams and individual parts.

Using the product in a manner not in accordance with the instructions may result in serious complications.

The product should be used according to the recommendations of the doctor or physical therapist.

Do not use the product if there are open wounds, skin abrasions, or if a warming gel or ointment has been applied.

Wearing the product at night is allowed only on the recommendation of the attending doctor.

Check the skin condition in the area where the product contacts, and if there are any skin changes, consult a doctor.

Unless otherwise recommended by a doctor, the product should not be worn for more than 4 hours per day.

WARNING:

In the event of a "serious incident" related to the product that directly or indirectly resulted in, could result in, or may result in any of the following events:

- a) death of the patient, user, or another person,
- b) temporary or permanent deterioration of the health of the patient, user, or another person,
- c) serious risk to public health,

this "serious incident" must be reported to the manufacturer and the competent authority in the country where the user or patient resides. In Poland, the relevant authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

WARNING:

If pain, allergic reactions, or other concerning, unclear symptoms occur related to the use of the medical product, consult a healthcare worker.

WARNING:

It is prohibited to use the product in a manner other than as intended.

CLEANING

The abdominal belt should only be hand washed in lukewarm water (up to 40°C) with a mild detergent (e.g., soap).

Do not iron. Do not dry clean. Do not use bleach. Protect from moisture. Do not expose to high temperatures. Do not tumble dry. Dry flat and gently press before drying.

STORAGE

Store the product in a dry place, away from direct sunlight.

DISPOSAL

After the product is decommissioned, it can be disposed of as regular household waste. Please protect the environment and follow the waste sorting norms and guidelines.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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